Instructions for use

Pump & Save™ Breastmilk Bags

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Mode d'emploi

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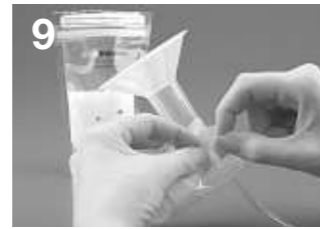
EN

Pump & Save™ Breastmilk Bags

Ideal to collect, store and freeze breastmilk

Collecting Breastmilk

- The label area allows you to write information on the bag.
- Gently open the zip-top closure, taking care not to tear the perforated top.
- Place the bag around the collar of the breastshield.
- Release the yellow strap by peeling where indicated, and unfold.
- Wrap the strap over the top of the breastshield.
- Peel the paper tab off the end of the strap and stick to the opposite side of the bag. Caution: Keep the paper tab away from children (danger of swallowing).
- Begin pumping. Do not fill the bag past the 150 ml mark.
- When you have finished pumping, remove the bag by tearing off the perforation just above the label area. The pleated bottom of the bag will allow it to stand up safely by itself while you put away your pump.
- To remove the strap from the breastshield, tear the strap and discard the top portion of the bag.
- Flatten the bag above the milk to remove excess air, then zip to close.

Hygiene

Wash hands thoroughly with soap and water before touching the breast and pump parts. Avoid touching the inside of the breastmilk bag. Clean the breast and surrounding area with a damp cloth. Do not use soap or alcohol which can dry the skin and cause cracked nipples.

Storing / thawing breastmilk

(For Healthy Term Babies)

Freshly expressed breastmilk	
Room Temperature	Do not leave at room temperature 3 – 5 days at approx. 4° C
Refrigerator	3 – 5 days at approx. 4° C
Refrigerator Freezer Compartment	6 months at approx. –16° C
Deep Freezer	12 months at approx. –18° C
Thawed breastmilk (previously frozen)	
Room Temperature	Do not store 10 hours
Refrigerator	10 hours
Refrigerator Freezer Compartment	Never refreeze
Deep Freezer	Never refreeze

- Do not store breastmilk in the door of the refrigerator.
- In order to preserve the components of the breastmilk, thaw in the refrigerator overnight or by holding the bag under running warm water (max. 37° C).
- Gently shake the bag to blend any fat that has separated.
- Do not thaw frozen breastmilk in a microwave oven or in a pan of boiling water (danger of burning).

Material:
Outer Layer: Polyester
Inner Layer: Polyethylene

Disposal:
According to local regulations
CE-mark according to 93/42/EEC

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Medela AG, Medical Technology