Instructions for Use SoftCup™ Advanced Cup Feeder

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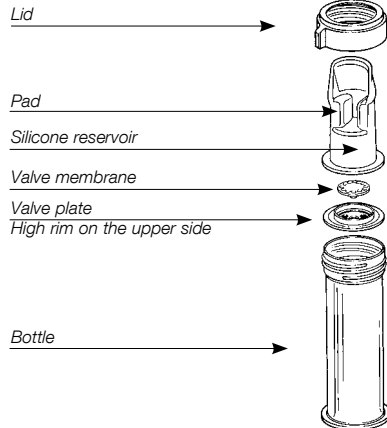
EN

SoftCup™ Advanced Cup Feeder


The SoftCup Advanced Cup Feeder facilitates feeding babies with sucking difficulties. It is an excellent alternative to cup feeding.

Assembly

1. Press membrane onto valve plate (upper side), so that the stud goes completely through the central hole.
2. Pour required amount of milk into the bottle. The milk should be at room temperature.
3. Put assembled valve across the mouth of the bottle (make sure that the membrane and high rim of valve plate are facing upwards).
4. Place the reservoir over the valve plate and screw all the parts together with the lid.

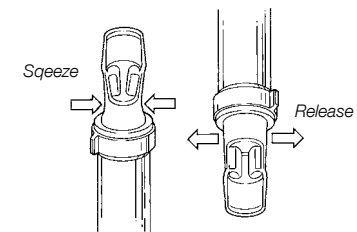Filling the Reservoir

1. Hold the feeder upright and remove some air from the reservoir by squeezing below the pads.
2. Keep squeezing and tip the feeder upside down.

3. Release the pads and some milk will enter the reservoir.
4. Repeat filling steps 1–3 until the reservoir is almost filled, approximately up to the bottom of the pads. As baby is fed, the reservoir will automatically refill by squeezing just below the pads.

Feeding the Baby

1. Hold feeder horizontally, with a slight angle upwards on the bottom of the bottle. Positioning is very similar to holding a pencil. Angle the feeder so the air that remains rises to the end of the bottle. Angling the bottle in this manner is essential to the automatic refill feature of the reservoir.

2. Begin the flow of milk by gently squeezing the two pads on both sides of the reservoir. This will push the milk in the feeding tip of the reservoir.
3. Use the tip of the reservoir to tickle baby's mouth open. Lay the rim of the reservoir on baby's lower lip.
4. Tip the feeder so that the milk pooling in the reservoir is fed to baby.

The milk should be lapped by the baby and with encouragement this will automatically happen. Please seek consultation when using this product and follow recommendations from professionals.

Cleaning and Sanitizing the Feeder

Before first use and once daily

The disassembled parts of the feeder should be boiled for 5 min., preferably in distilled water. When used in hospitals all disassembled parts can be autoclaved to a maximum of 134 °C for 3 (or 18) min. or 125 °C for 15 min. To avoid damage during sterilization, sterilize parts without applying external pressure on parts.

After each feeding

If there is some milk left in the feeder, hold it nearly horizontal over a sink before unscrewing the lid. The valve should remain in the reservoir, to prevent

milk from spilling. Take the feeder apart, making sure that the valve is separated from the reservoir. Flush cold water through the slit in the pads of the reservoir to remove as much milk residue as possible. Wash all parts in warm soapy water. Rinse thoroughly with cold water. Do not turn the reservoir inside out or push brushes, etc. through the slit in the reservoir.

Helpful Hints

Reservoir not filling properly

Make sure the slit in the pad is open. Use stronger finger pressure to help the flow of milk.

Reservoir not refilling automatically

If thickened supplements are used, check that the air-groove in the disk has not become blocked.

Feeder leaking

If the feeder leaks, check that the feeder has been correctly assembled. The stud of the valve must be properly inserted through the hole in the disk. The lid must be firmly screwed onto the bottle. Make sure the supplement is not too hot. The feeder is designed for use with supplement at room temperature and should not be used with hot liquids. Check to see if either the valve membrane or reservoir is visibly damaged and needs replacing.

Material

Reservoir and valve membrane: Silicone
Other parts: Polypropylene

Disposal

According to local regulations.

Medela also offers a Disposable Baby Cup Feeder. It is available in sets of 10 pieces.

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