
EN	Instructions for use	2
ES	Manual de instrucciones	11
PT	Manual de instruções	20
EL	Οδηγίες χρήσης	29
HE	הוראות שימוש	46

I Harmony™ Breastpump

Mother's milk is the best nutrition for babies. Harmony gives you and your baby optimal support during the breastfeeding period. The attractive manual breastpump is especially suitable for occasional pumping at home or on the go.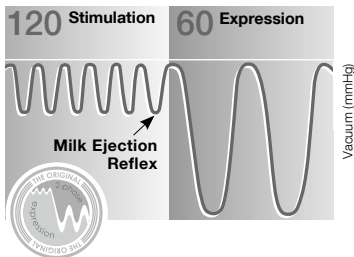

Harmony has **2-Phase Expression®**, the only research based breastpump technology to mimic a baby's natural nursing rhythm.

Stimulation:

Fast pumping rhythm to start your milk flowing.

Expression:

Slower pumping rhythm to express your milk gently and efficiently.

Proven to feel most like your baby and to reduce your pumping time.

II Indications For Use

The Harmony Breastpump is a manual breastpump:

- To be used by lactating women to express and collect milk from their breast during absences from the baby (e.g. an evening out from baby, working part-time).
- To relieve engorgement occasionally.
- For home use and on the go.

III Important Safeguards


Read all instructions before use.

- Use the product only for its intended use as described in this instruction manual.
- **Use Medela original accessories only.**
- Check pump kit components for wear or damage.
- This is a single user product. Use by more than one person may present a health risk.
- If problems or pain occur, consult your lactation consultant or physician.

SAVE THIS INSTRUCTION MANUAL

IV Assembly Instructions

1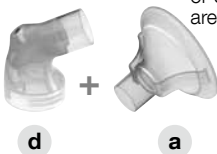

Make sure breastshield (PersonalFit or SoftFit Breastshield) and connector are securely attached.


2

Snap membrane onto valve until membrane lies completely flat.

IV Assembly Instructions

3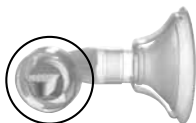

Push assembled valve and membrane onto connector. Ensure sidewise position of valve as indicated on picture.

f g + d a

4

Screw bottle into connector.

a d f g


h

5

If the handle components have been disassembled, place the oval shaped base of the stem into the diaphragm.


6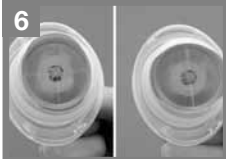



Make sure the oval shape is lined up correctly in the diaphragm.

Correct

Incorrect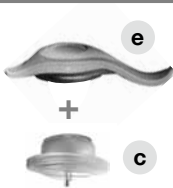

Place diaphragm and stem into pump handle and press down firmly around entire edge to ensure proper seal.

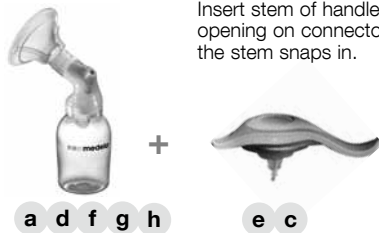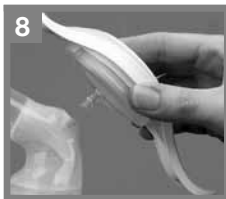Insert stem of handle assembly into opening on connector. Make sure that the stem snaps in.

V Pumping

Be comfortably seated and relax. Use a footrest, when pumping or when feeding your baby. Elevating your feet puts you into proper sitting position to avoid stress on the lower back and perineum.

Center your breastshield over your nipple.

Do not tilt or overfill bottles when pumping.

V Pumping

2

Start pumping with **Stimulation** by pressing down on the milk drops symbol on top of the handle. Press and release quickly to imitate a baby's fast suckling until your milk starts to flow or you sense milk ejection.

3

Once your milk starts to flow or you sense milk ejection, change to **Expression** by pressing the lower end of the handle. This will give you a longer stroke that imitates a baby's slower, deeper sucks. The level of vacuum is controlled by the compression and the speed of the handle stroke.

Once finished pumping disassemble the pump kit items. Close the bottle and store it for later use.

Use bottle stand to prevent tip over.

VI Breastshield Fitting

Choosing the right size of Breastshield

A correctly fitted breastshield avoids compression of any milk ducts during pumping to ensure optimal breast emptying and maximum milk output.

Your breastshield is the right size if your answer to the following questions is «Yes»:

- Is your nipple moving freely in the tunnel?
- Is minimal or no areolar tissue being pulled into the tunnel of the breastshield?
- Do you see a gentle, rhythmical motion in the breast with each cycle of the pump?
- Do you feel the breast emptying all over?
- Is your nipple pain-free?

If your answer to any of the above questions is «No», try a larger (or smaller) breastshield.

Medela PersonalFit™ Breastshield: Available in five sizes – because one size does not fit all.

VII Cleaning and Hygiene

Follow these cleaning instructions unless you are told otherwise by your healthcare professional.

Important

- Use drinking water quality for any of the following described cleaning procedures.
- Clean parts immediately after use to avoid dry up of milk residues and to prevent from growth of bacteria.
- Disassemble all parts prior to cleaning. Especially be sure to remove the yellow valve head from the breastshield and separate the white valve membrane from the valve head.

Prior to first use and once daily

- Disassemble, wash and rinse all parts of your pump kit that come in contact with the breast and milk (a/b+d+f+g+h+j+k+l).
- Place parts in a saucepan of sufficient capacity.
- Fill with cold water until the parts are well covered. Optionally, a tea spoon of citric acid may be added to avoid a build-up of limescale.
- Heat it up and allow to boil for 5 minutes.
- Remove from stove and drain water off.
- Dry parts with a clean towel.

Alternatively, the disassembled parts can be sanitized in top rack of dishwasher or steam-sterilized.

Note: Food colorings may discolor components which will not affect performance.

After each use

- Disassemble and handwash all parts of your pump kit that come in contact with the breast and milk (a/b+d+f+g+h+j+k+l) in warm, soapy water.
- Rinse in clear water.
- Dry parts with a clean towel.

Hints

After cleaning, when parts are not in use, put in a clean zip lock bag or store in a container with lid. Alternatively, the parts can be wrapped in a clean paper or cloth towel.

VII Cleaning and Hygiene

Hygiene

- Wash hands thoroughly with soap and water before touching the breast and pump parts.
- Avoid touching the inside of bottles and lids.
- Clean the breast and surrounding area with a damp cloth.
- Do not use soap or alcohol which can dry the skin and cause cracked nipples.

VIII Storing / Thawing Breastmilk

Check with your hospital for specific instructions.

Breastmilk Storage (For Healthy Term Babies)

	Room Temperature	Refrigerator	Refrigerator Freezer Compartment	Deep Freezer
Freshly expressed breastmilk	<i>Do not leave at room temperature</i>	<i>3 – 5 days at approx. 4° C</i>	<i>6 months at approx. -16° C</i>	<i>12 months at approx. -18° C</i>
Thawed breastmilk (previously frozen)	<i>Do not store</i>	<i>10 hours</i>	<i>Never refreeze</i>	

- *When freezing do not fill bottles/bags more than 3/4 full to allow space for expansion.*
- *Label bottles/bags with the date of pumping.*
- *Do not store breastmilk in the door of the refrigerator.*
- *In order to preserve the components of the breastmilk, thaw in the refrigerator overnight or by holding the bottle/bag under running warm water (max. 37° C).*
- *Gently shake the bottle/bag to blend any fat that has separated.*
- *Do not thaw frozen breastmilk in a microwave oven or in a pan of boiling water (danger of burning).*

IX Troubleshooting

If there is low or no suction

Make sure

- the breastshield forms a complete seal around the breast.
- all connections of the kit and breastpump are secure.
- the white valve membrane is laying flat against the yellow valve head. Check that the valve head and membrane are clean and not damaged. See «Assembly Instructions».

X Disposal

The unit is made of plastics and may only be disposed of in accordance with local regulations.

XI Parts List

Article numbers

a	SoftFit™ Breastshield (24 mm)	810.0402
b	PersonalFit™ Breastshield (24 mm)	810.7084
c	Diaphragm/stem with O-ring	800.0766
d	2-Component Connector	800.7196
e	Handle	800.7170
f	Valve head	810.0475
g	Valve membrane	820.7013
h	Bottle (150 ml)	811.0021
i	Bottle stand	810.0462
j	Cap	810.0474
k	Lid	810.7062
l	Disc	810.0473

*Does not contain Bisphenol-A, DEHP (Phthalate) or Latex.
Subject to change.
Content may vary from picture.*

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