Instructions for use

Quick Clean™ Microwave Bags

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Quick Clean™ Mikrowellen-Beutel

Mode d'emploi

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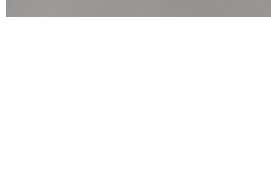
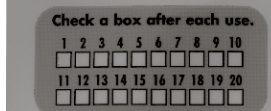
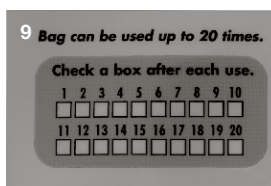
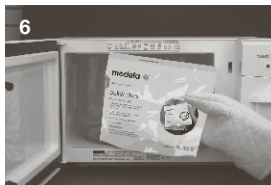
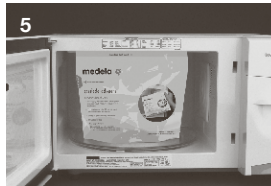
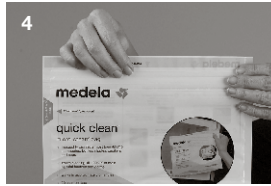
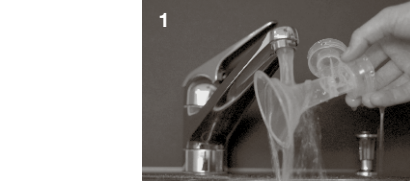
Quick Clean™

Microwave Bags

- Use drinking water quality for any of the following described cleaning procedures.
- Clean parts immediately after use to avoid dry up of milk residues and to prevent from growth of bacteria.
- Follow these cleaning instructions.

One bag will hold

- 1 breastshield, connector, valve, membrane, milk bottle and cap or
- 2 breastshields, connectors, valves and membranes or
- 2 milk bottles and caps or
- Breastpump tubing

1. Disassemble and thoroughly wash all parts in warm, soapy water. Rinse in clear water.

- | | | | |
|-----|--|-----|---|
| 2a. | Put either <ul style="list-style-type: none"> • 1 breastshield, connector, valve, membrane, milk bottle and cap or • 2 breastshields, connectors, valves and membranes or • 2 milk bottles and caps | 2b. | Lay breastpump tubing flat on bottom of bag. Do not place any other items in the bag with the tubing. |
|-----|--|-----|---|

- | | | | |
|-----|---|-----|---|
| 3a. | For breastshields, connectors, valves and membranes, milk bottles: Add 60ml water to bag. | 3b. | For breastpump tubing: Add 120 ml water to bag. Make sure tubing is covered with water. |
|-----|---|-----|---|

- Do not overfill bag.
- Using less than 60 ml/120 ml water may warp the parts.

4. Seal bag tightly. Put bag into microwave on center of turntable. Do not place the bag on the metallic grill that comes with some microwave ovens.

1.5 minutes at 1100 W+	3 minutes at 800 – 1100 W	5 minutes at 500 – 750 W
------------------------	---------------------------	--------------------------

Heating for more than the recommended time may disfigure plastic parts. If your microwave does not have a turntable, rotate bag halfway through.

6. Using oven mitts or hot pad, carefully remove bag from microwave oven.

7. Before opening bag, remove water by pouring out through steam vent.
Caution
When removing bag from microwave or opening bag, hot water or steam may escape (danger of burning).

8. Remove parts from bag and dry them with a clean towel. After cleaning, when parts are not in use, put in a clean zip lock bag or store in a container with lid. Alternatively, the parts can be wrapped in a clean paper or cloth towel.

9. Bag can be used up to 20 times. Check a box after each use.

Safeguards

- Bag intended for use with microwave-safe plastics only.
- Piston and cylinder of Medela Manual Breastpump cannot be cleaned in microwave.

Note

- Steam cleaning may discolor components or may cause clouding in the tubing. This will not affect performance.
- Do not place the bag in a conventional oven or in an oven grill.
- Follow the instructions for your microwave oven.

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